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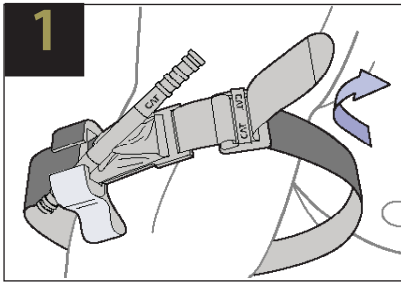
### LIMITATION OF LIABILITY:

C-A-T Resources, LLC, its parent company Composite Resources, Inc., its employees, agents, contractors, suppliers, and distributors shall assume no liability for injury or damages arising from the application and use of the Combat Application Tourniquet® (CAT). The user assumes all risk of liability. The CAT should only be used as directed by user's military service component guidelines, EMS authority, or under the supervision of a physician.

### STANDARD USE GUIDELINES

When applied in accordance with directions, the Combat Application Tourniquet® (CAT) is a safe and effective device for controlling life-threatening extremity bleeding. If you cannot be sure or cannot take the additional time to examine where the bleeding is coming from based on the situation, the CAT can be effectively applied over clothing as high on the arm or leg as possible. The CAT must NOT be applied over solid objects within the clothing. As soon as the situation permits, the injured limb should be evaluated and the CAT re-positioned 2"-3" above the injury directly to the skin.

Combat Application Tourniquet® and CAT are trademarks registered in the United States and certain other countries.

U.S. Patent No. 7,842,067; 7,892,253;  
Canada Patent No. 2,569,550;  
EU Patent No. 1,753,344;  
Germany Patent No. 602005031888.7;  
Israel Patent No. 179769;  
Other Patents Pending

Manufactured by C-A-T Resources, LLC  
483 Lakeshore Pkwy, Rock Hill, SC 29730  
United States of America  
803.325.9300

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CAT Resources, LLC